

TOILET SEAT CUSHION

CSH1061



To see all FAQs in one place, visit [vhealth.link/w4a](https://vivehealth.link/w4a)

vive
vivehealth.com

QUICK START GUIDE

INSTALLATION

1. Clean the toilet seat thoroughly and ensure it's completely dry.
2. Undo the velcro straps.
3. Place the cushion on the toilet seat, aligning it properly.



4. Press down firmly on the cushion to engage the 6 suction cups and non-slip bottom



5. Lift the toilet seat and secure the velcro straps:
 - a. Loop each strap under the toilet seat.
 - b. Thread the strap through the D-ring.
 - c. Pull tight and press the velcro together to secure
6. Lower the seat and sit on the cushion to ensure it's properly attached and comfortable.

REMOVAL

1. Lift the toilet seat and undo the velcro straps, removing them from under the toilet seat.



Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com

250924

2. Lower the seat and use the included putty knife to carefully break the suction of the cups:
 - a. Insert the putty knife between the cushion and toilet seat.
 - b. Gently pry upwards to release the suction.



3. Lift the cushion off the toilet seat.

CLEANING AND MAINTENANCE

1. Wipe down with a disinfectant spray or cleaning solution.
2. Dry thoroughly before reattaching to the toilet seat.
3. For deep cleaning, remove the zippered cover and hand wash or machine wash on gentle cycle.
4. Allow the cover to air dry completely before reassembling.



****DO NOT MACHINE DRY!****

SAFETY NOTES

- Always inspect the cushion and straps for wear before each use.
- Ensure the cushion is securely attached before sitting.
- If you experience any discomfort, discontinue use and consult a healthcare professional.

For more information or customer support, please visit vivehealth.com or call **1-800-487-3808**.