

TRANSFER SLING STANDARD

LVA3097BLK

vive
vivehealth.com



To see all the FAQ's
in one place,
visit vivehealth.link/5q7

OWNER'S MANUAL

TRANSFER SLING STANDARD

WHAT'S INCLUDED

- 1x Transfer Sling Standard
- 4x Removable Pads



KEY WARNINGS

- NEVER exceed 250 lb. weight capacity.
- Minimum of two caregivers required.
- NEVER leave patient unattended during transfer.
- DO NOT use with uncooperative patients.
- Always inspect before use; discontinue if damaged.

QUICK START GUIDE

1. Pre-Transfer Preparation.

- Explain procedure to patient.
- Ensure 2+ caregivers are present.
- Clear path and position surfaces close together.
- Lock wheels on beds/wheelchairs.
- Attach all 4 cushions to handles.



2. Application & Transfer

- Position sling under patient (while seated or by rolling patient if lying down).



- Center properly with bottom of transfer sling at tailbone and top near mid back.



- Verify all four handles are accessible on both sides.



- Using proper body mechanics (bent knees, straight back), **lift simultaneously** to transfer patient.



3. Post-Transfer

- Ensure patient comfort and stability.
- Remove sling when safe.
- Check patient's skin.
- Clean if necessary (spot clean only, air dry).

CONTRAINDICATIONS

Do not use if patient:

- Exceeds 250 lb weight limit.
- Is combative or uncooperative.
- Has open wounds that could contact sling.

CARE INSTRUCTIONS

- Remove handles before washing
- Machine wash cold, gentle cycle
- Air dry only
- Do not bleach

WARRANTY

This Transfer Sling is backed by a 6 month limited warranty from the date of purchase. This warranty covers defects in materials and workmanship under normal use.

If the product proves to be defective during the warranty period, we will, at our discretion, repair or replace the unit free of charge. Proof of purchase is required for warranty service.



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at
[vhealth.link/5q7](https://vivehealth.com/faq) for helpful answers.



And if that doesn't answer your question, our
customer service team would love to help!
Feel free to connect with them by phone,
e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
Made in China