

TRI-ROLLATOR WITH SEAT

MOB1116

vive®
MOBILITY




To see all FAQs in
one place, visit
vivehealth.link/zk8

vivehealth.com

TRI-ROLLATOR WITH SEAT

WHAT'S INCLUDED

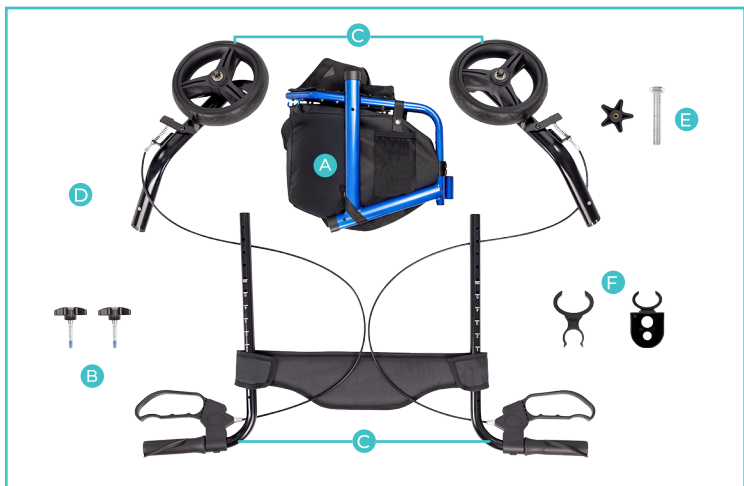

- (A) 1x Tri-Rollator Frame w/ Bag
- (B) 2x Hand Screws
- (C) 2x Rear Legs attached to 2x Upright Handles
(Attached by the brake line)
- (D) Front Wheel
- (E) 1x Star Lock with Hex Screw
- (F) Cane Holder Clip on Pieces

ASSEMBLY INSTRUCTIONS

Want to watch us do this instead? Just go to the following link to see how to assemble and set up your Tri-Rollator with Seat.



To see all FAQs in one place, visit **vhealth.link/9vu**

Step 1

Unfold the Rollator Frame by pulling the two sides of the frame apart.



Step 2

Set up the (A) Frame and center it in the middle of the (C) 2 Rear Legs attached to 2 Upright Handles as seen in the first picture below. Make sure the legs point inward towards the frame and the brake lines are untangled and flow straight upward towards your Upright Handles.




Then insert each leg into the rear of the frame. If done correctly the wheel will be on the outside of the frame with the brake line on the underside leading upward towards the metal friction brake heading towards the top side of the tire.




Step 3

Install the Front Wheel (D) by inserting it into the opening on the front of the frame. Then take the (E) Hex screw and place it in the hex hole opening on the inside of the frame, thread it through so it protrudes outward from the frame. Lock it into place with the Star Lock.

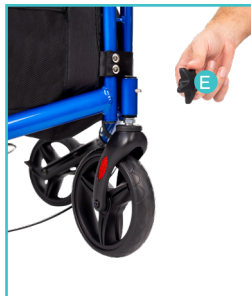

Step 4

Follow the Brake Cable from one of the Rear Wheel Assemblies to the attached Hand Grip Assembly. Fit the Hand Grip down into the hole on top of the frame above the connected Rear Wheel Assembly.

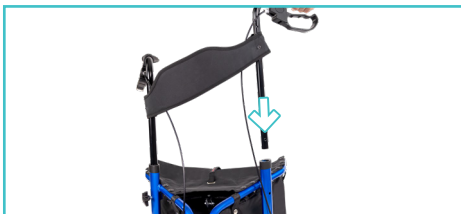

Step 5

Set the Hand Grip to a comfortable height where the holes in the socket and the grip bar are aligned, and secure it in place with a Hand Screw.




Step 6

Repeat Steps 4 and 5 with the other Hand Grip Assembly on the other side of the frame. Make sure the Hand Grips are both set to the same height.

Step 7

Add the two (F) Cane Holder clip on pieces to the frame as seen in the pictures to the right.

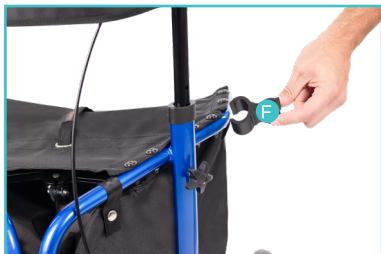

Step 8

Before use make sure the rollator is in the lock position.

Locking the frame for use:

Pull the locking strap from under the seat, pull it all the way out making sure it has fully clicked forward locking the bars just slightly past a 180 degree angle.



Unlocking the frame for folding:

Pull back the unlock strap on the seat.



USING THE BRAKES

There are two (2) different brake functions on the Rollator: an operating brake and a parking brake.

The operating brakes are intended to help maneuverability around corners or other objects, as well as provide a somewhat firm base for leaning on when needed. These brakes are engaged and released using the brake handles on the handlebars. Each side controls the brake for the corresponding rear wheel (left for left, and right for right). Just squeeze the brake handle to engage the brake on the corresponding wheel.



The parking brake is used to help prevent the Rollator from moving when not in use, or to prevent it from moving while sitting or standing. To engage the parking brake, push the parking brake handle down until it locks into a diagonal position, locking it into place. To disengage the parking brake, simply lift up on the handle.



To fold the Rollator for easy storage or transport, empty or remove the Frame Bag and pull back on the black knob at the folding hinge on the front of the Rollator. Then grab the two (2) sides of the Rollator frame and press them together to collapse it.

MAINTENANCE

The only maintenance required for your Tri-Rollator is to keep it clean with a soft, damp cloth. Do not use any abrasive or volatile cleaners.

TROUBLESHOOTING

Occasionally, the brake cables can become loose. To tighten them, simply use a phillips head screwdriver to tighten the adjuster screw where the cable is connected to the brake handle. DO NOT attempt to use the Rollator if the brake cables are broken.

If you have any problems with the Rollator, including assembly, use, or maintenance, please contact service@vivehealth.com. Please report to vivehealth.com if any unexpected events occur.

ADDITIONAL INFORMATION AND SAFETY WARNINGS ⚠

- The Rollator has a weight limit of 300 lbs, and should not be used to support more weight than that.
- The Rollator is intended as a mobility aide and should not be used for any other purpose. Failure to follow this warning can result in damage to the equipment and/or severe injury.
- Do not attempt to use the Rollator if either of the brake cables are loose, disconnected, or broken.
- The Rollator is intended for adult use only. Children should not attempt to use this equipment.
- Do not store the Rollator outside or in areas of extreme moisture, cold, or heat as these can damage the components over time, making them unsafe to use.
- Do not use this rollator unless you are sure the Locking strap is pulled tight and locked into place.
- Do not attempt to sit in the seat unless the brakes are locked.

WARNING: No modification of this equipment from its original specifications is allowed.



HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at [vhealth.link/zk8](https://vivehealth.com/health.link/zk8) for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
Made in China