

FINGER SPLINT

SUP3008



To see all the FAQ's,
visit [vhealth.link/zwk](https://vivehealth.link/zwk)

vive
vivehealth.com

QUICK START GUIDE

APPLICATION

1. Position onto the injured finger.
2. Secure the bottom strap followed by the top.



QUICK START GUIDE



Distributed by



8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com

CARE

Hand wash with a mild detergent in cold water. Hang dry.

WARNINGS

- Consult your doctor before use.
- Stop use if pain occurs or worsens.
- Assistance may be required during the application of this support.
- Do not over tighten, doing so may lead to restrict blood circulation.

- If you experience any pain, swelling, sensation changes, or any unusual reaction while using this product please consult you doctor immediately.
- Do not machine wash or dry the splint.