Professional Massager

Commercial-Use Therapeutic Massager

IMPORTANT SAFEGUARDS

When using an electrical appliance, basic precautions should always be followed, including the following:

READ ALL INSTRUCTIONS AND SAFEGUARDS BEFORE USING.

DANGER

To reduce the risk of death or electric shock:

1. Do not reach for an appliance that has fallen into water. Unplug immediately.
2. Do not use while bathing or in the shower.
3. Do not place or store appliance where it can fall or be pulled into a tub or sink. Do not place in or drop into water or other liquid.
4. Always unplug this appliance from the electrical outlet immediately after using and before cleaning.
5. Do not use massager without attachments.

WARNING

To reduce the risk of burns, fire, electric shock or injury to persons:

1. This appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts.
2. Do not operate this appliance under a blanket or pillow. Excessive heating can occur and cause fire, electric shock or injury to persons.
3. Close supervision is necessary when this appliance is used by, on, or near children, or disabled persons.
4. This massager contains an electro magnetic motor. To avoid risk of injury, it must not be used within six inches of an implantable cardiac device.

5. Use this appliance only for its intended use, as described in this manual. Do not use attachments not recommended by the manufacturer.
6. Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a Service Center for examination and repair.
7. Do not carry this appliance by supply cord or use cord as a handle.
8. Keep the cord away from heated surfaces.
9. Never drop or insert any object into any opening on this appliance.
10. Do not use outdoors.
11. Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
12. In case of pregnancy, diabetes or illness, consult your physician before using this massager.
13. Do not use massager on:
 - a. Sensitive skin, swollen or inflamed areas, in the presence of poor circulation, where skin eruptions are present, or in the presence of unexplained calf or abdominal pain.
 - b. Varicose veins.
 - c. The front of the neck.
 - d. Genital areas.
 - e. Sleeping or unconscious persons.
14. When using a massage attachment, be sure to seat the attachment fully onto the stem and keep it on while massaging.

15. Do not use this massager for more than 20 minutes at one time.
16. To disconnect, turn switch to the OFF position, then remove plug from outlet.
17. Do not use massager in close proximity to loose clothing or jewelry.
18. Keep long hair away from massager while in use.

SAVE THESE INSTRUCTIONS For Commercial Use Only


CORD

If your massager has a polarized plug: To reduce the risk of electric shock, this appliance has a polarized plug (one blade is wider than the other). This plug fits in a polarized outlet only one way. In the event the plug does not fit fully in the outlet, reverse the plug. In the event it still does not fit, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

OPERATING INSTRUCTIONS

1. With the massager switch in the OFF position, plug the power cord into an AC outlet.
2. Slide the switch to the desired massaging speed — LOW or HIGH.
 - **I - LOW** — for soothing massage.
 - **II - HIGH** — for vigorous, deep-penetrating action.

MASSAGE ATTACHMENTS

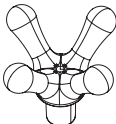FOUR FINGER FLEX ATTACHMENT

Soft, finger-like nodes replicate a human touch massage.

GENERAL BODY ATTACHMENT

For general, all-body massage.

SCALP

Delivers effective scalp massage, loosens dandruff, and is soothing and relaxing.

CHANGING ATTACHMENTS

To remove an attachment head, use your thumb and two fingers, and pull away from the massager stem. To insert an attachment head, align the head with the massager stem, and press firmly onto stem.

CAUTION: The two holes in the stem of the massager are like an electrical plug — electrical power is transmitted through them. **NEVER** allow water near the stem, and never insert anything into the holes.

CLEANING/STORAGE

Always unplug massager before cleaning.

To clean, simply wipe unit with a dry, soft cloth. Do not immerse in water or use any harsh cleaners or solvents. When storing unit, make sure cord is not twisted or kinked.

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LIMITED 1 (ONE) YEAR WARRANTY

If your Wahl product fails to operate satisfactorily within one (1) year from the date of original purchase or receipt as a gift, Wahl will at our option repair or replace it and return it to you FREE OF CHARGE. Our warranty does not cover blades which are wear out parts. Please mail your product prepaid to Wahl Clipper Corporation, 3001 N. Locust Street, Sterling, Illinois 61081, Attention: Repair Department, with a note describing the defect and the date of original purchase or receipt as a gift (if in warranty). Products out of warranty will be repaired at our standard repair charges.

This warranty is void and no repair or replacement will be made under this warranty or otherwise if:

1. The product is modified in any manner.
2. The product is repaired by anyone other than Wahl Clipper Corporation
3. The product has been subject to unreasonable use.

This warranty gives you specific legal rights and you may also have other rights which may vary from state to state. No other written express warranty is given. Any implied warranty including any warranty of merchantability, which may arise from purchase or use is limited to the one-year period provided in this express warranty. Some states may not allow such limitation, so it may not apply to you. No responsibility is assumed for incidental or consequential damages of any type, or for the use of unauthorized attachments. Some states may not allow the exclusion or limitation of incidental or consequential damages, so the limitation or exclusion may not apply to you. Warranty valid only in the United States.

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